

OrthoPulse™

LAVAGE SYSTEM

Splash shield included.

Non-clogging suction design.

Spray stem may be removed
for use as extended suction.

Fracture-resistant spray stem and suction tip.

PVC- and Latex-free construction.
Environmental friendly materials.

Polyurethane hose preventing blocked saline flow.

OrthoPulse Lavage System provides:
optimal cleaning power,
pulse frequency, pulse strength
and saline flow rate.

Airtight construction minimizes
the risk of contaminated air
leaking into the sterile field.



Advantages:

Superior Materials

Flexible and fracture resistant spray stems and suction tips. Reduces risk of intraoperative breakage. All disposables are made out of PVC-free and Latex-free materials.



Superior Design

Excellent, non-clogging suction design ensures smooth passage of debris.

Superior Power

OrthoPulse Lavage System provides: optimal cleaning power, pulse frequency, pulse strength and saline flow rate. Resulting in superior preparation of bone surface and cleaning of wound site.



Less Waste

Instead of disposing the entire lavage system only the nozzles are disposed. Resulting in reducing contaminated waste by more than 80% compared to a battery powered pulse lavage system.



Instructions:



1) Read instructions.

Connect airhose to handpiece.

2



3) Set regulator to 5-7 kg/cm² (80-100 psi).

Connect disposable set to handpiece.

4



Attach vacuum line (optional for suction sets).

5



Connect spike tip to saline bag.

6

Ready to use.

7



Clamp irrigation tubing when not in use.

8



Pulse Lavage

OML9000	Orthopulse Handpiece Zimmer connector
OML9010	Orthopulse Handpiece AO connector
FL1014	Orthopulse Airhose
FL1015	Orthopulse Airhose MA7 connector

Disposables

OMP9000	Orthopulse Hip Set Standard
OMP9001	Orthopulse Knee Set Standard
OMP9100	Orthopulse Hip Set Suction
OMP9101	Orthopulse Knee Set Suction
OMP9200	Orthopulse Soft Tissue Set
OMP9201	Orthopulse Soft Tissue Long Set
OMP9300	Orthopulse Small Bone Set
OMP9400	OrthoPulse Narrow Hip Set Suction
OMP9801	Orthopulse Suction Tubing, 20 pcs.
OMP-100-091	Orthopulse Clip, one pair, sterile

Accessories

FL1602	Orthopulse AO Washing Plug
FL1611	Orthopulse SZ Washing Plug

Manufactured for:

Pulse Lavage Ltd.

P.O. Box 16142

CY-2086 Nicosia

CYPRUS

E-mail: pulselavage@pulselavage.com

Phone: +357-24-427 117

Web: www.pulselavage.com

Manufactured by:

Oy Fluorplast AB

Västervägen 8

66240 Petalax

FINLAND



0537